INDICATION

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50 ml tube fitted with a canula

INDICATION

Filmogen liquid bandage for the treatment of superficial bedsores and chronic wounds.

COMPOSITION

Glycerol (66.52%), Honey, Aqua, MMP-cyanidins (derived from extracts of *Camellia sinensis* leaves and *Vaccinium myrtillus* fruit), Xanthan gum.

MODE OF ACTION

Wound healing implies allowing new cells to grow and multiply in order to repair the injured tissues. Normal cell growth can only take place in a clean, chemical-free, specific protein-free, contaminant-free, hydrated & aerated environment. When applied onto the surface of chronic wounds, ANTISCAR forms a highly osmotic, protective, flexible and resistant film over the lesion. This hypertonic film mechanically attracts hypotonic fluid present below the injured surface. This creates a strong and continuous outward flow of hypotonic liquid that cleans the injured surface by detaching and draining contaminants, bacteria, viral particles, dead cells and free-floating protein molecules. A clean, hydrated and contaminant free environment instantly stimulates cell growth and healing. ANTISCAR acts topically, mechanically, without being absorbed in the body and protects the injury against irritation and external aggressions. Clinical results show significant reduction of wound surface and depth within a short period of time.

DIRECTIONS

Wash hands. Clean the wound surface with a cotton gauze soaked in physiological saline or a gentle wound cleanser. According to the size of the injury, pour a sufficient amount of ANTISCAR solution directly onto the wound so as to form a thin film over the entire surface. We suggest protecting the wound by covering it with a clean and breathable cotton gauze bandage allowing aeration. Wipe the tip of the tube with a clean tissue after use. Close the tube with the plastic cover after each use. For young patients, product should be applied by an adult or at least with adult supervision and/or assistance.

When beginning the treatment, apply twice a day for non-exuding wounds or 3-4 times a day for exuding wounds. Then at least once a day. Interrupt the treatment for at least 2 days after each period of 30 consecutive days of use.

PRECAUTIONS

Read the instructions leaflet carefully before use and respect the directions. This product is for topical use on damaged skin surface only and should not be swallowed. Keep the product out of reach and sight of children. Wash hands before and after use. Avoid contact between the tip of the tube and the lesion during application. To reduce the risk of cross infection, each tube must be used by one patient only. Seek medical advice if you see no improvement within 3 to 4 weeks of use of this product, or in case of worsening of symptoms and/or associated fever. Do not use after the expiry date, more than 30 days after 1st use or if the tube is damaged. ANTISCAR is not toxic if swallowed but may cause slight temporary digestive disorders related to concentration in glycerol. Avoid contact with the eyes. In case of contact, flush thoroughly with water.

STORAGE

Store at room temperature, do not freeze or refrigerate.

INTERACTIONS

Due to the product's composition and to its mechanical & topical mode of action, the probability of interaction with any systemic or topical treatment is unlikely. However, we suggest not to use any chemical or cytotoxic treatment simultaneously with ANTISCAR.

CONTRAINDICATIONS

Check the product's composition and do not use if you are sensitive or allergic to any of the ingredients. Although all the ingredients used to prepare this product are derived from food ingredients and used in very small quantities, in the absence of sufficient specific safety data, do not use this product during pregnancy or breastfeeding without medical advice and in children below the age of 12.

SIDE EFFECTS

In certain cases, a slight to moderate local irritation may be observed following application but will disappear after a few minutes. If you observe any type of allergic reaction after product application (ex: strong persistent irritation/ burning sensation), cease using the product, flush the wound surface thoroughly with clear fresh water, and consult a medical professional if necessary. If you notice any serious incidents due to the use of this product, please notify immediately the manufacturer/distributor and national competent authorities.

The packaging (including the container) can be disposed of as household waste after use.

Leaflet version / revision: AS b7-0621
VACN0384

Year of affixing of the CE marking : 2008

Manufacturer :
VITROBIO SAS
ZAC de Lavaur
63500 ISSOIRE, France

Distributor :
Naturveda SAS
ZAC de Lavaur
63500 ISSOIRE, France
information@naturveda.fr