MIG SPRAY

MIGRAINE

PRÉVENTION

NATURVEDA

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PREVENTION

NATURVEDA

15 ml filmogen nasal spray
(approx. 140 sprays)

INDICATION Migraine prevention

COMPOSITION


Aqua, Glycerol (9.10%), blend of Acacia and Xanthan gums, Migcyanidin : association of *Salix alba*, *Curcuma longa*, *Vitis vinifera*, *Mentha piperita* and *Tanacetum parthenium* extracts; potassium sorbate, sodium benzoate, citric acid.

MODE OF ACTION

When applied on the nasal mucosa as a spray (2-3 sprays), (1) the product forms an osmotic, non-irritant, absorbent film, that remains stable for 4 to 6 hours on the surface of the nasal mucosa (NM). (2) Osmosis generated by the glycerol base, instantaneous and mechanically, attracts the contaminants and undesired proteins, such as CGRP, TNF- α , IL-6, & IL-12, involved in triggering of migraine attacks which are present on the surface & deep parts of the mucosa, towards the film. (3) These molecules are then retained into the film. (4) The osmotic flow cleans nasal surface & allows cell regeneration. (5) The elimination of migraine-specific proteins continues throughout the duration of the film's action, minimizing their concentration and the resulting impact on headache severity. (6) The osmotic film acts locally. The osmosis generated maintains the separation between the nasal mucosa (NM) and the film. The MIG SPRAY film acts mechanically, without contact with the nasal mucosa, and is therefore classified as a medical device (MD) in Europe. (7) This mode of action justifies the absence of any addiction. (8) Clinical results show a significant reduction in the frequency and intensity of migraine attacks and an improvement in quality of life.

DIRECTIONS

1. Thoroughly clean your nose before using the product.
2. Shake the bottle.
3. Remove the protective cap.
4. Press the sprayer until a fine mist of product appears.
5. Insert the nozzle into one nostril and press on the sprayer to apply 2-3 sprays in each nostril while breathing in slightly.
6. Clean the nasal insert with tissue paper and close the bottle with the cap after each use.

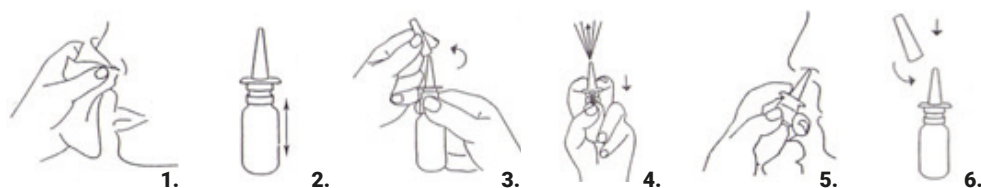Apply 3-4 times per day.

PRECAUTIONS

The product is intended for topical nasal use only. Read the instructions carefully before use and follow the directions. Wash hands before and after use. Use each bottle for one patient only to reduce the risk of cross-infection.

The mode of action of MIG SPRAY is due to instant osmosis and may induce slight tingling/burning sensation during the first 10-15 minutes after use. However, if the irritation is too severe and/or persists over time with the same intensity, you may be allergic to one of the ingredients and we advise you to stop using the product. The product may cause a strong nasal discharge within 10-15 minutes of application. Avoid blowing your nose heavily and wipe off the excess with a clean tissue.

Interrupt the treatment for at least 4 days after each period of 30 consecutive days of use.

STORAGE

Keep out of the reach and sight of children. Store at room temperature (do not refrigerate or freeze). Do not use after the expiry date, more than 30 days after opening or if the bottle is damaged.

INTERACTIONS

Due to the product's composition and to its mechanical & topical mode of action, the probability of interaction with any systemic treatment is unlikely. For best results, do not apply any other topical nasal treatment 15 minutes before or after the use of this product.

CONTRAINDICATIONS

Check the list of ingredients and do not use the product if you are sensitive or allergic to any of those ingredients. Although all the ingredients used to prepare this product are derived from food ingredients and used in very small quantities, in the absence of sufficient specific safety data, we do not recommend the use of this product in pregnant or breastfeeding women without medical advice or in children below the age of 12. Do not use in patients allergic to salicylates.

SIDE EFFECTS

MIG SPRAY is non-toxic if accidentally swallowed. Avoid contact with eyes. In case of contact, rinse with plenty of water. If you experience an allergic reaction within minutes of applying the product (burning, swelling, strong and persistent irritation, etc.), stop using it, rinse your nose with fresh water, and consult a health professional if necessary. If you notice any serious incident related to the use of this product, please notify immediately the manufacturer/distributor (by visiting www.naturveda.fr/materiovigilance) and the competent authority of your member state.

After use, the packaging items (including the bottle) can be disposed of as household waste.

Leaflet version / revision: MIG b1-0321

Year of affixing of the CE marking : 2021

Manufacturer :
VITROBIO SAS
ZAC de Lavaur
63500 ISSOIRE, France

Distributor :
Naturveda
ZAC de Lavaur
63500 ISSOIRE, France

Medical Device
Made in France